

Specimen Collection Swab

Instruction for Use

Product

Specimen Collection Swab

Model and Specification

Specification: 100 pieces/Box, Model: N150, O150.

Model	Total Length	Brush Head	Handle
N150	152±4 mm	22±3 mm	82±4 mm
O150	152±4 mm	22±3 mm	31±3 mm

Composition

Swab (brush head and handle). Sterile. Disposable.

Parts


No other accessories.

Intended Use

For sample collection. N150 is applicable for sample collection from nasal cavity. O150 is applicable for sample collection from oral cavity.

Instruction

1. Take out the swab from its package to start the sampling.

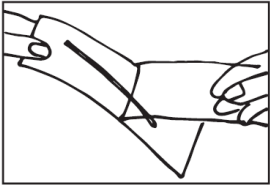


2. After collecting the sample, place it into the specimen tube containing the preservation solution.



3. Break the extra brush handle at the mouth of the tube to leave the brush head in the specimen tube.



4. Tighten the tube cap and indicate the number and date.

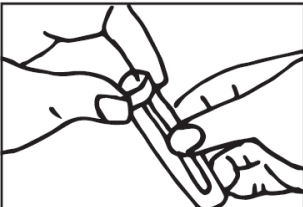

Contraindications

Use with caution for those allergic to flocking.

Precautions

1. For single use only. Do not reuse. Dispose it immediately after use. The disposal of its wastes shall be carried out in accordance with the relevant national laws and regulations on environmental protection.
2. Do not use if the package has been damaged or the product has expired.
3. Please read the instruction manual carefully before use, pay attention to the use-by date and batch number of the product.

Package

When the product leaves the factory, the inner packaging is paper-plastic packaging, and the packaging form is sealed packaging. The outer packaging is a corrugated carton, with the product certificate and product instruction manual.

Storage and Transportation

Storage: Store in a well-ventilated room with relative humidity no more than 80%, and no corrosive gas.

Transportation: The packaged products can be transported by common shipping methods, and should be handled with care during transportation to avoid direct sunlight, violent collision, rain and gravity.

Shelf Life

24 months.

REF	Catalogue number		Temperature limit
	Consult instructions for use	LOT	Batch code
IVD	In vitro diagnostic medical device		Use-by date
	Manufacturer		Do not re-use
	This way up		Keep dry
	Fragile, handle with care		Do not use if the package is damaged